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DESCRIPTION:

Temporary sharps holder with multiple access points.

INDICATIONS FOR USE: The ShortStop temporary sharps holder with multiple access points is intended to help prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION: Do not reuse.

CAUTION: All needles should be flushed prior to use.

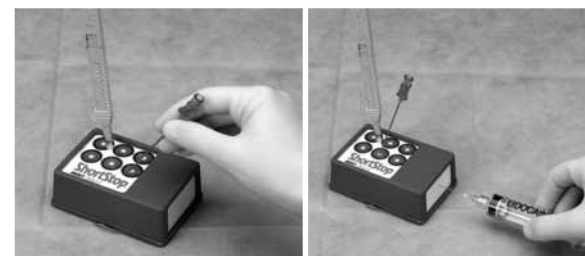
CAUTION: When inserting sharps into the temporary sharps holder, do not support the device with your hands as inadvertent needle sticks may occur.

CAUTION: Needle bevel contact with the plastic wall of the device, may result in damage to the needle.

INSTRUCTIONS FOR USE: Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop Advantage in the sterile field with the printed foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.

2. After use, the clinician may temporarily place a sharp device in either the top access compartment or side access compartment of the temporary sharps holder.

3. After completion of procedure, dispose of the sharp devices using universal precautions. Place the sharp devices in an OSHA compliant waste disposal system. After the sharp devices have been removed from the foam, place the used ShortStop Advantage device in a hospital compliant waste disposal system.

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