**INSTRUCTIONS FOR USE**

**PRODUCT DESCRIPTION**

The One-Step is a short term drainage catheter with introducer needle and vent plug.

**INTENDED USE**

The One-Step Centesis Drainage Catheter is intended for the percutaneous drainage of fluids.

**CAUTIONS**

- Read instructions prior to use
- **Rx Only Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician.
- Contents of unopened, undamaged package are sterile using ethylene oxide.
- Do not leave in patient longer than 24 hours.
- Store in a cool dry place
- Only persons knowledgeable of the risks involved and qualified in the procedures performed should use this device.

**REUSE PRECAUTION STATEMENT**

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or reesterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or reesterilization may also create a risk of contamination of the device and/or cause patient infection or crossinfection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

**INSTRUCTIONS FOR USE**

Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Remove catheter and needle from protective boot.
2. Insert catheter into fluid collection site using standard insertion technique.
3. When catheter is in appropriate position remove needle from catheter.
4. Attach device with standard luer fitting to catheter hub to remove fluid.
5. Following the procedure gently withdraw the catheter and dress the site accordingly.

Manufacturer: [www.merit.com](http://www.merit.com)  
Merit Medical Systems, Inc. South Jordan, Utah 84095  
U.S.A. 1-801-253-1600  
U.S.A. Customer Service 1-800-356-3748

**EC REP** Authorized Representative:

Merit Medical Ireland Ltd, Parkmore Business Park West,  
Galway, Ireland

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