


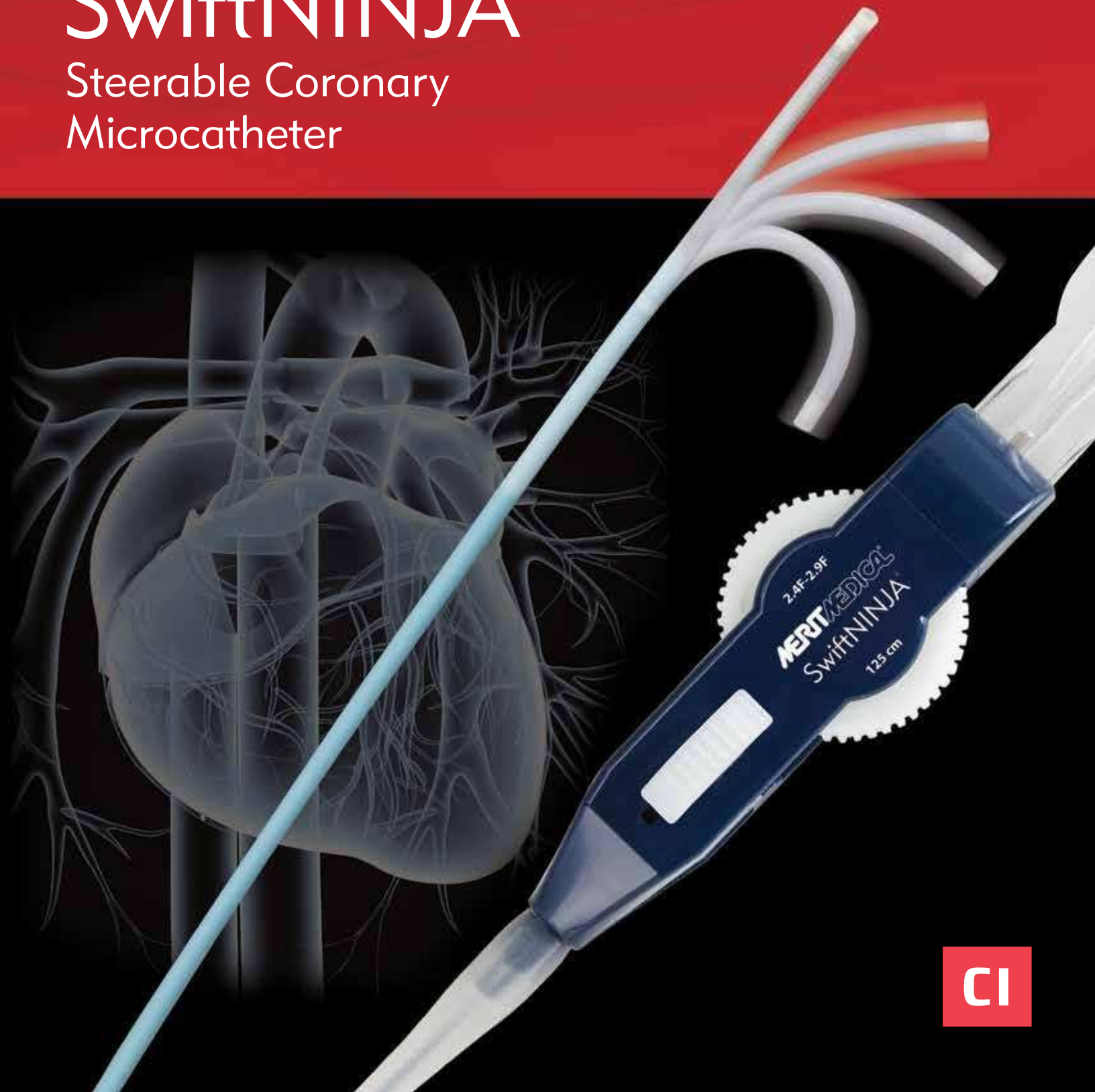

SwiftNINJA[®]

Steerable Coronary
Microcatheter




SwiftNINJA[®]

Steerable Coronary Microcatheter


SwiftNINJA[®]

The most advanced and only 180° articulating coronary and peripheral vascular microcatheter on the market today. Speed, control, support and navigation at your fingertips.

Hydrophilic coating
on distal 80 cm

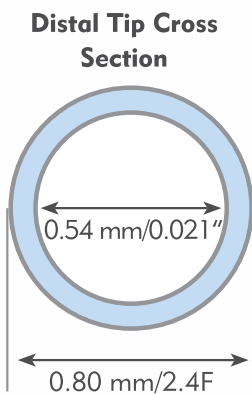

Close-up of Steerable Tip

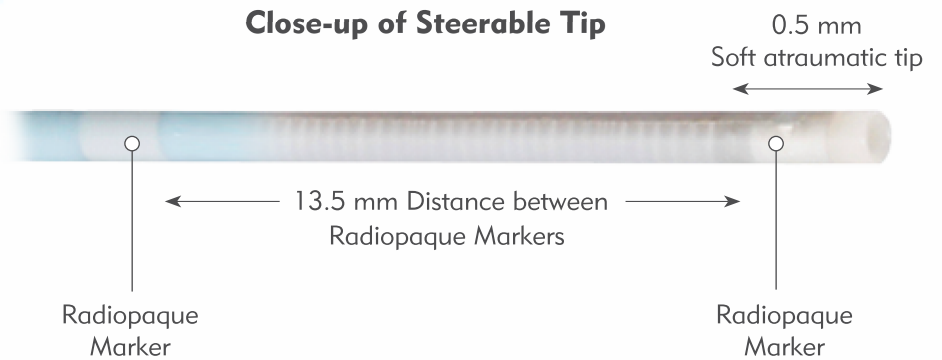

Fluoroscopic Visualization


Two radiopaque marker bands
for enhanced navigation and
tip placement

INNOVATIVE DESIGN

Innovative microtechnology designed to articulate in opposing directions to navigate the most challenging vascular anatomy.


Steering Dial Mechanism

Allows opposing 180° articulation from neutral

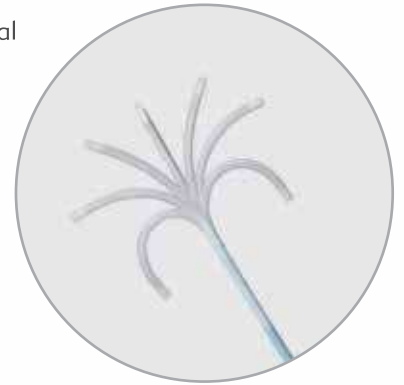

Steering Lock

Once shaped, the tip can be locked in position

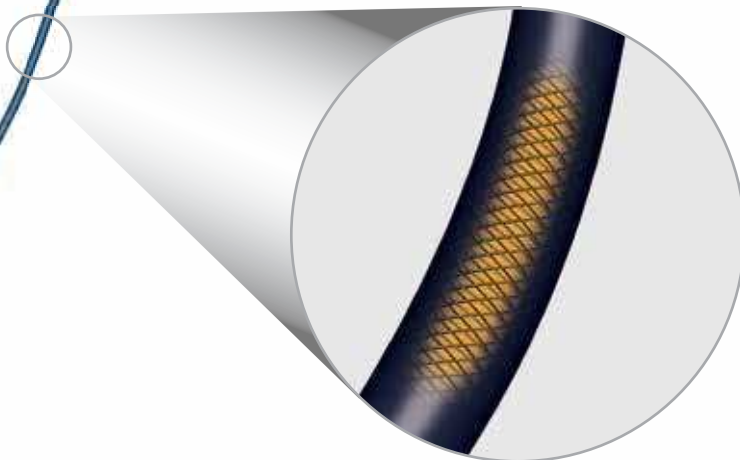

Tungsten-braided Shaft

Braided shaft enhances shaft support and provides pushability

SwiftNINJA®

Steerable Coronary Microcatheter



Catheter Specifications

Part Number	Inner Diameter	Outer Diameter Proximal Portion	Outer Diameter Distal Portion	Usable Length	Hydrophilic Coating Length	Max Guide wire	Maximum Injection Pressure	Distance From Tip To 1 st Markerband	Distance Between 1 st And 2 nd Markerbands	Cath Vol.
SNCM125	0.021" (0.54 mm)	2.9F (0.97 mm)	2.4F (0.80 mm)	125 cm	80 cm	≤ 0.018" (0.46 mm)	6,900 kPa (1,000 psi)	0.5 mm	13.5 mm	0.49 mL

Catheter Preparation for Use

To activate the hydrophilic coating: Flush the catheter with heparinized saline via the standard luer BEFORE removal from the protective catheter sleeve.

To Unlock Steering Dial (white wheel): Hold the lower housing body and gently pull the steering dial toward the luer connector until a clicking sound is heard.



Prior to attempting catheter tip movement or deflection, you must engage the steering dial

Ordering Information:

Part Number	Description
SNCM125	SwiftNINJA® Steerable Coronary Microcatheter 2.9F-2.4F 125 cm



Before using, refer to Instructions for Use for indications, contraindications, warnings, precautions and directions for use.



Understand. Innovate. Deliver.™

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.208.4300
1.800.35.MERIT

merit.com

Merit Medical Europe, Middle East, & Africa (EMEA)
Amerikalaan 42, 6199 AE Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733

Australia
+61 (0) 3 8373 4943

Austria
0800 295 374

Belgium
0800 72 906 (Dutch)
0800 73 172 (Français)

Brazil
+55 11 4561 - 0062

Canada
1.800.364.4370

China
+86-10 8561 0788

Denmark
80 88 00 24

Finland
0800 770 586

France
0800 91 60 30

Germany
0800 182 0871

Hong Kong
+852-2207 6888

India
+91-80-41223376

Ireland (Republic)
1800 553 163

Italy
800 897 005

Korea
+8210 2205 0707

Luxembourg
8002 25 22

Netherlands
0800 022 81 84

Norway
800 11629

Portugal
308 801 034

Russia
+7 495 221 89 02

Spain
+34 911238406

Sweden
020 792 445

Switzerland
(Deutsch)
+41 22 518 02 30

(Français)
+41 22 518 02 52

(Italiano)
+41 22 518 00 35

UK
0800 973 115