TEMNO

Evolution[®]

EN	INSTRUCTIONS FOR USE	2
FR	MODE D'EMPLOI	3
IT	ISTRUZIONI PER L'USO	4
DE	GEBRAUCHSANWEISUNG	5
ES	INSTRUCCIONES DE USO	6
PT	INSTRUÇÕES DE UTILIZAÇÃO	7
PTBR	INSTRUÇÕES DE USO	8
NL	GEBRUIKSINSTRUCTIES	9
SV	BRUKSANVISNING	10
DA	BRUGSANVISNING	11
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	12
TR	KULLANMA TALİMATLARI	13
PL	INSTRUKCJA UŻYTKOWANIA	14
CS	NÁVOD K POUŽITÍ	15
BG	ИНСТРУКЦИИ ЗА УПОТРЕБА	16
HU	HASZNÁLATI ÚTMUTATÓ	17
RU	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ	18
FI	KÄYTTÖOHJEET	19
ZH	使用说明	20
JA	使用説明	21

INSTRUCTIONS FOR USE

DEVICE DESCRIPTION


The TEMNO Evolution® biopsy system is a single patient use core biopsy device. It is available in several needle gauge sizes and lengths. The plunger is color coded according to the various gauge sizes (e.g., green=20G, orange=18G, blue=16G, white=14G)."

INDICATIONS FOR USE

The TEMNO Evolution® biopsy device is used to remove, by cutting, a specimen of tissue for microscopic evaluation.

STEP 1. COAXIAL TEMNO PLACEMENT


During the pre-procedure set-up, insert the biopsy device through the coaxial introducer needle to verify compatibility. Adjust sliding stopper on coaxial needle to desired calibration level. Lock in place. Insert coaxial and stylet until sliding stopper rests on skin surface. Image coaxial needle to make sure the tip is on the surface of the lesion. Remove inner stylet.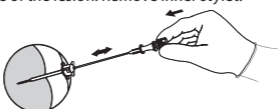

STEP 2. TEMNO NEEDLE SPRING CHARGE


If using a 10 mm Notch Exposure, apply the spacer onto the TEMNO needle and press fit firmly onto luer taper of TEMNO. Charge TEMNO needle by pulling back on plunger until firm click is felt indicating that the needle spring is charged. It is possible to load the needle in two different positions.

- First Click = 10 mm Notch Exposure (Use Spacer)
- Second Click = 20 mm Notch Exposure (Do not use Spacer)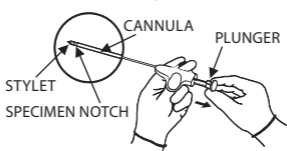

STEP 3. TEMNO NEEDLE PLACEMENT

With stylet fully retracted, (i.e. specimen notch completely covered by the cannula as in STEP 2) insert TEMNO needle into coaxial needle. Press fit coaxial needle hub onto luer taper of Temno or luer taper of spacer for 10 mm notch.

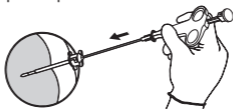
STEP 4. TEMNO NEEDLE EXECUTION

Press plunger with thumb to advance specimen notch into area to be sampled. Image again to insure that notch is inside the lesion. Press plunger past stop with thumb, automatically triggering the cannula to close over the notch trapping the specimen.

STEP 5. TEMNO NEEDLE EXECUTION

Draw needle out. Pull back on plunger to recharge the spring. Push plunger to first stop exposing the specimen notch. Remove tissue specimen from notch in stylet. Be careful not to push plunger past first stop during this step.

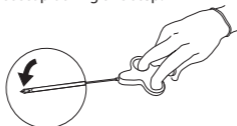CAUTION: For single use only. Re-use may result in a non-functional product or contribute to cross contamination. Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

WARNING

- Not intended for use in bone.
- Do not use if package is damaged.

SYMBOL	DESIGNATION
	Use By: YYYY-MM-DD
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Do Not Use if Package is Damaged
	Single Use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
	Caution: Consult accompanying documents

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Merit Medical Systems, Inc.
South Jordan, Utah 84095 U.S.A.
U.S.A Customer Service 1-800-356-3748
Merit Medical Ireland Ltd,