



Needleless catheter securement device,

with easy viewing, protection and cleaning.





SECUREMEN

A revolutionary new device that saves time, provides convenience and enhances compliance with safety regulations by securing catheters with a needleless procedure.



STEP ONE PEEL off the paper backing.

STEP FOUR

TWIST back and forth

to remove finger rings.



STEP TWO CENTER and APPLY Revolution to skin that is clean, dry, and free of any solvents. Pass the catheter through the center of the Revolution and apply to the skin.



STEP THREE **PULL FINGER RINGS to** RELEASE the first set of strings until they are snugly deployed around the catheter. Do not pull hard, this can kink or cut the catheter.



clockwise to deploy the next set of strings. Stop rotation when strings wrap snugly around the catheter. DO NOT OVER TIGHTEN. Place your thumb at the catheter tie-down post prior to

rotating the blue ring. This allows for easier turning and removes

pressure from bandage.

ROTATE THE BLUE RING

STEP FIVE



TIE off bottom strings around catheter using a surgeon's knot. DO NOT OVER TIGHTEN. This may cause damage to the catheter or restrict flow.

ONE DEVICE FOR **VARIOUS SIZE CATHETERS**

For drainage catheter sizes 6F - 24F.

CENTRAL ADHESIVE HYDROCOLLOID

Skin friendly, absorbent around puncture site, conforms well to body anatomy.

NON-WOVEN MATERIAL

with ribbed coated adhesive secures to skin for up to 7 days with comfortable adhesive.



COVER LID Provided to cover the puncture site while showering for added protection.

STEP SIX

ORDERING INFORMATION

CFG624

Box qty. 25 units Supplied sterile, single pouch. Intended for one-time use.

LATEX FREE



Merit Medical Systems, Inc. • 1600 West Merit Parkway, • South Jordan, Utah 84095 • 1-801-253-1600 • 1-800-35-MERIT Merit Medical Europe • Horsterweg 24 • 6199 AC MAASTRICHT- AIRPORT • The Netherlands • +31 43 35 88 222 Merit Medical Ireland Ltd. • Parkmore Business Park West • Galway, Ireland • +353 (0) 91 703 733 Free phone from specific country:

www.merit.com