

STEPS FOR APPLICATION

STEP ONE



REMOVE paper backing. **CENTER and APPLY** Revolution to skin that is clean, dry, and free of any solvents. Pass the catheter through the center of the Revolution and apply to the skin.

STEP THREE




ROTATE the BLUE RING clockwise to deploy the next set of strings. Stop rotation when strings wrap snugly around the catheter. **DO NOT OVER TIGHTEN.** Place your thumb at the catheter tie-down post prior to rotating the blue ring. This allows for easier turning and removes pressure from bandage.

COVER LID



Merit recommends having the lid in place when showering.


STEP TWO

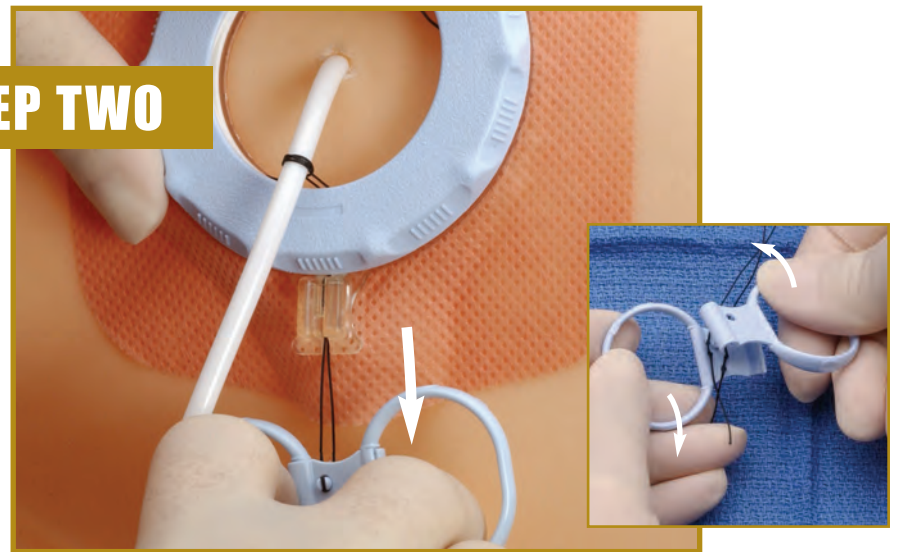

PULL FINGER RINGS TO RELEASE the first set of strings until they are snugly deployed around the catheter. **DO NOT PULL HARD, THIS CAN KINK OR CUT THE CATHETER.**

TWIST to remove finger rings.

STEP FOUR

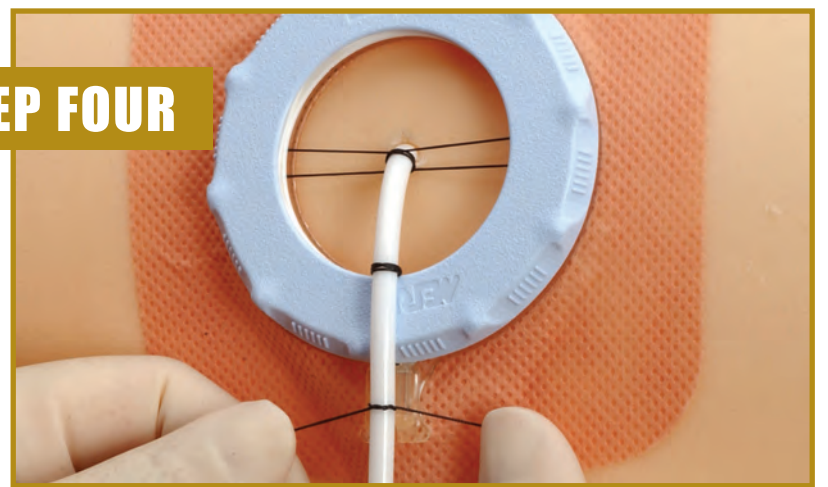

TIE off bottom strings around catheter using 3-4 knots. **DO NOT OVER TIGHTEN.** This may cause damage to the catheter or restrict flow.

REMOVAL



CUT STRINGS before removal.

The Revolution™ Catheter Securement Device is designed to secure a non-vascular catheter to help minimize catheter movement and accidental removal.

- The Revolution is designed to secure catheters from 6F to 24F. Before placing the Revolution, ensure that the skin is clean, dry and free of any solvents.
- The Revolution is designed to remain secure for up to 7 days. If longer securement is desired, a skin adhesive should be used prior to placing the device.

OTHER MERIT DRAINAGE PRODUCTS



ReSolve® Drainage Catheters



MDD600 Drainage Bag



VacLok® Syringes



One-Step™ Centesis Catheters

